

# ***italograph*® Micro**

## Frequently Asked Questions

Please select area of interest from one of the following headings:

[Power On](#)

[General](#)

If your query is not answered in this document then please click on the following link, fill in the details and submit the form. One of our technical support personnel will then reply to your query.

[Technical Support Request Form](#)

Device doesn't power on.

- Check battery and replace if required.
- Check that the battery is connected correctly and that there are no loose connections. The lead should be pushed back through the hole in the battery compartment.

Why does the device take a long time to be ready for testing?

- The device is zeroing its flow sensor. Please ensure that the device is not being moved excessively.
- Check flowhead.
- Ensure that the device has stabilised to room temperature.

What is the port inside the battery compartment for?

- This is for future enhancements.

Can I print or download test results from the Micro?

- No, the Micro will only display the results, as it is intended for quick, portable assessment without the complication of printers or external devices; Vitalograph do however produce [portable spirometers \(Vitalograph 2120\)](#) with these capabilities.

I am concerned about cross-infection – can I use a BVF with the Micro?

- Yes, but an adaptor is required – part no. 63051.